IMPLANT GUM HARD SCAN (AUTOMIX)

Stand: 12/04/18

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Stand: 12/0418

Instruction of Use

A-Silicone for construction of gingiva masks for CAD/CAM technology

For the manufacture of fixed partial dentures the morphology of the gingiva is an essential reference to prevent functional and aesthetic problems. M+W Implant GUM HARD Scan is a gingival mask for CAD/CAM technology based on **addition curing vinyl silicones** delivered in **AUTOMIX-cartridges**. It can be extruded directly and without any bubbles into the mould. Therefore a fast and simple manufacture of the gingival mask is possible.

M+W Implant GUM HARD Scan supplies the essential data to the dental technicians for the determination of the cervical contours and the extreme limits for preparation of the fixed dentures. Because of its great transversal strength M+W Implant GUM HARD Scan can be easily removed from the gypsum model without tearing. During the manufacture of the fixed denture, it is possible with M+W Implant GUM HARD Scan to locate and prevent the pressure onto the gingiva, which comes from the construction of the fixed denture. Therefore, it can prevent possible inflammations of the parodontium.

Gun and cartridge

Load the application gun with the AUTOMIX CARTRIDGE. Remove and dispose the closure-cap. For exact flow control extrude slightly material until uniform flow from both orifices is achieved. Install the mixing cannula onto the cartridge and extrude needed material.

Note:

Store used cartridge with fixed used mixing cannula. **Do not re-use the cartridge cap.** Closing the cartridge by turning on the cap between use could cause cross contamination of components and thus premature setting.

Preparation of the removable gingival masks

- a) The gypsum model has to be prepared as usual.
- b) With a suitable addition curing or condensation curing impression material with a high Shore-A hardness make an impression of the relevant part of the jaw. After completely setting of the silicone, the silicone impression has to be removed.
- c) The trimmed stumps of model are to set free under consideration of the extreme limits for preparation.
- d) The silicone impression, prepared according b) serves as mould in conjunction with the trimmed gypsum model for the preparation of the gingival mask. Within removing of the gingival contours it originates a space between the silicone impression and the trimmed model. This space has to be filled with M+W Implant GUM HARD Scan for preparation of the gingival mask. For preparation of the mould it has to be drilled two small holes side by side through the silicone part of the mould. One hole is a channel for filling of the M+W Implant GUM HARD Scan, the other one serves as a channel for escaping of the air in the mould. For an accurate and easy separation between the gingival mask and the silicone part of the mould, the silicone mould should be wetted inside with a suitable separator liquid (M+W Implant GUM Separator) if a condensation-curing silicone was used. If **addition-curing silicone** was used for preparation of the silicone mould, it is **necessary** to wet the silicone mould inside with M+W Implant GUM Separator.
With help of the mixing cannula extrude so much material of M+W Implant GUM HARD Scan until the first material comes out through the other hole. The working time (**23 °C, 74 °F**) of M+W Implant GUM HARD Scan is **1:30 minutes**; the setting time inclusive working time (**23 °C, 74 °F**) is **about 7 - 8 minutes**. After removing of the silicone impression the gingival mask remains on the trimmed model, which references the form of the gingival borderline. Based on the prepared mask on the model the prosthetic work can be start.

Storage

Do not store above 25 °C (78 °F). Do not use after expiry date.

Additional note:

Do not use Latex gloves, they may interfere with the setting reaction of the impression material.

Warranty

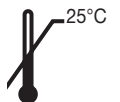
P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior Dental Materials GmbH product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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Manufacturer: P.L. Superior Dental Materials GmbH · 22607 Hamburg · Germany