Ⓓ Anweisungen

Produktname

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10 Stück/Blister, 30 Blister/Schachtel

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Packing

10 pieces/blister, 30 blisters/box

This product is made of stainless steel wire and is used for the treatment of dental pulp and surrounding tissues.

Usage method

- If the affected tooth is a vital pulp, please obliquely insert the short barbed broaches along the canal wall to reach 2/3 of the root canal depth and rotate clockwise 90° and pull

it out, completely pulling out the strip-shaped formed pulp; if the pulp has decomposed and necrotized, please insert a small amount of root canal irrigation into tooth cavity first. Then use the short barbed broaches to extract the pulp in stages from shallow to deep, and simultaneously use the irrigation to shake and wash multiple times until all debris is removed.

- The small hole at the end of the handle can be used to fasten a safety chain for the short barbed broaches, which can avoid accidentally falling into the trachea

Precautions

- The short barbed broaches should not be forcefully pushed in the root canal when encountering resistance. Please insert the root canal slowly and tentatively, not pushing forcefully, which can prevent the instrument from breaking or being wedged into the root canal and unable to be removed. The rotation angle should be less than 180°.
- Avoid multiple repeated use.

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