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GB Instructions

Test for the daily check of the disinfection and cleaning performance of the thermal disinfectant. Place in a tray or container every day for the first wash cycle of the thermal disinfectant using the stainless steel test holder (for multiple use).

Application:

Place a wash test strip in the stainless steel test holder, with the red test fields facing upwards.

The strip must be inserted such, that test field 1 lies in the covered area and test base 2 lies in the visible range.

Perform a complete wash cycle. After completing the wash cycle, take the test strip from the test holder and compare the result with the references illustrated below.

Condition:

Cleaning and disinfection machine: i.e. Miele, Melag etc. – Programme: 60~93°C/ 10 min.

Positioning:

in the middle of the stand.

Note:

The standard reference is based on our own in-house data. Wash conditions, for example, the max. performance, depend on the model and year of manufacture of the cleaning and disinfection machine. Please use these instructions as a guideline for checking the daily wash performance of your machine, which is subject to daily fluctuations, during the daily process.

If fluctuations in wash performance are observed, we recommend the following:

1. Check the washing conditions, such as detergent flow, blocking in the spray arms, as well as overloading, etc.
2. Check the protein substance residue immediately in your own system.

Initial position prior to wash cycle

Acceptable values after wash cycle

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