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*Triton frei

INKTEST EASY

Sealed seam leak test for the routine testing of sealed seams according to DIN EN ISO 11607-1 Appendix B & ASTM F1929

Content: Box with 30 x 3-5ml test pouch

(GB) Instructions for use

Sealed seam leak test for the routine testing of sealed seams according to DIN EN ISO 11607-1 Appendix B, as well as ASTM F1929*. Irregularities in the welded seams (channeling etc.) become visible immediately after emptying the disposable pipette. No other aids are required. According to the standard, see above, routine checks are to be performed by the operator.

Please always wear gloves! Protect your worktop with an appropriate underlay! We will not be liable for any possible contamination.

- Prepare an approximately 25cm long piece of sterilization foil with a width of at least 150mm.
- Now seal one end of sterilization foil.
- Place a pouch on the foil. (Figure 1)
- Now carefully seal the open side of the foil without exerting any pressure on the pouch (Figure 2).
- Now press with the flat of your hand on the pouch until it bursts open. (Figure 3)
- Allow the test liquid to react on each side of the sterilisation foil for a maximum of 5 seconds (by tightening and rotating the foil - 4-sided test, Figure 4).

You should now **not** be able to detect any channeling or similar irregularities (visual inspection). Dispose of the foil filled with ink carefully in a waste container.

The same can be performed with already prefabricated sterilisation pouches.

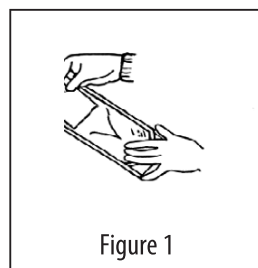
Figure 1

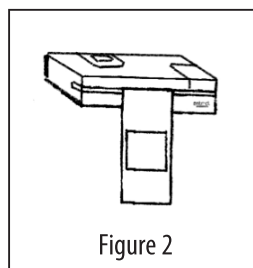
Figure 2

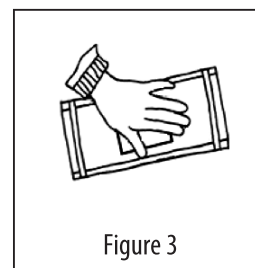
Figure 3

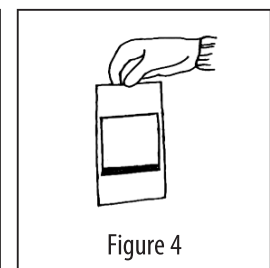
Figure 4

First Aid Measures

Skin contact: Wash thoroughly with water and soap. Consult a physician in case of skin irritations.

Eye contact: Rinse immediately for 10-15 minutes with opened eyelids with running water. Then consult a physician.

Swallowing: Rinse mouth thoroughly. Never administer anything to unconscious persons by mouth. Do not induce vomiting without instructions from a physician. In case of complaints, please consult a physician.

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Müller & Weygandt GmbH · Reichardsweide 40 · 63654 Büdingen/Germany · Tel.: +49 (0) 60 42 - 88 00 88