PERMABOND SE

Stand:02/1019

D Gebrauchsinformation

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GB Instructions for use

M+W Permabond SE is a light-curing, self-etching one component adhesive system. It can be used for all restorations with light-curing composites, compomers and ormoceres.

Recommended use

Prepare cavity, pulp protection and base lining according to the instructions for use of the filling material. Clean cavity with water spray and dry with oil-free air. Dentin must not be over-dried!

1. Put 2-3 drops of M+W Permabond SE on a mixing pallet. Apply on tooth with single-use brush and gently work into tooth structure for 10 sec. Let material set for more 20 sec.

2. Blow and dry material at least 5 sec. to remove solvent. Pay special attention to still have sufficient material on tooth.

3. Light cure adhesive with LED of halogen curing unit for 10 sec.

4. Filling material may be applied onto cured M+W Permabond SE immediately.

Please note

• A suitable pulp/ dentin protection is to be used to prevent irritation of pulp. Areas close to pulp have to be covered with a thin layer of a suitable underfilling such as calcium hydroxide. Areas, which are not covered with calcium hydroxide, are to be treated with M+W Permabond SE.

• M+W Permabond SE is easily inflammable. Keep away from possible ignition sources. Measures against electrostatic charge have to be taken. Fume of the product should not be inhaled.

• M+W Permabond SE contains methacrylate,

which can lead to skin and eye irritations. In case of contact with eyes flush eyes with a lot of water and if necessary seek medical attention.

• Contamination of the content of the bottle caused by contaminated single-use brushes is to be avoided.

Interactions

Materials containing eugenol can inhibit polymerization of M+W Permabond SE. Therefore do not use zinc-oxide cements in connection with M+W Permabond SE.

Contraindications

Applications is contraindicated for direct application on the opened pulp or when allergies against any components of M+W Permabond SE are known.

Side effects

Side effects are not known to date. In singular cases,

hypersensitivity to components of the material cannot be excluded.

Composition

M+W Permabond contains BIS-GMA based dental resins, ethanol, water, additives and catalysts.

Classification

No relevant norm.

Storage

Store dry and cool between 2°C – 8°C/ 36 – 46°F. Avoid direct sunlight. Please close bottle immediately after use. Do not use after the expiration date.

Product sizes:

REF 000 181 Pack containing 5 ml fluid

Keep away from children! For dental use only!

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indigodental GmbH · Fahltskamp 5 · D-25421 Pinneberg