# PERMABOND LAB NEW FORMULA

Stand: 18/2023

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## Instructions for use

### MDP containing primer for bonding of methacrylate based dental materials

#### Product description

Permabond Lab is a primer to improve the adhesion between methacrylate based dental materials to titanium, non precious and precious metal alloys. Based on innovative MDP technology Permabond Lab allows an excellent adhesion.

#### Indications/Intended use

Bonding of bonding of methacrylate based dental materials to metal alloys (all types)

#### Performance features

The performance features of the product meet the requirements of the intended use.

#### Contraindications

In case of known allergy to components of Permabond Lab.

#### Patient target group

Persons who are treated during a dental procedure.

#### Intended users

This medical device should only be used by professional trained dental/dental technician users.

#### Application for indirect restorations

##### Preparing the surface to be faced

Sandblast the surface to be faced with aluminium oxide (50-110 µm) at 2-3 bar pressure. Then dry with clean compressed air.

##### Application

Apply a uniform layer of Permabond Lab with a brush on the surface to be faced and leave on for 10 seconds. Air thin gently to remove the volatile components. Apply a methacrylate based dental material to the prepared surface according to the corresponding instructions.

##### Storage

Do not store above 25 °C (77 °F)! Do not use after expiry date.

**Permabond Lab is highly volatile. Close bottles immediately after use.**

##### Additional Notes/Warnings

- Permabond Lab contains Acetone.
- Acetone is highly volatile. Use only in good ventilated rooms.
- Avoid contact with eyes and skin.
- In case of contact with eyes, rinse immediately thoroughly with plenty of water and consult an ophthalmologist.
- In case of skin contact wash with water.
- Keep away from children!

##### Composition

Acetone, MDP.

##### Disposal

Disposal of the product according to local authority regulations.

##### Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

##### Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

##### Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior Dental Materials GmbH product.

##### Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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P. L. Superior Dental Materials GmbH · Stiller Weg 15 A · 22607 Hamburg/Germany
**M+W Dental** Müller & Weygandt GmbH · Reichardsweide 40 · 63654 Büdingen/Germany · Tel.: +49 (0) 60 42 - 88 00 88