GLASFILL APLICAP

Stand 280622

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FEATURES

A fast setting, high strength glass ionomer restorative with excellent aesthetics. Radiopaque. The cement adheres directly to tooth structure and is especially suited to minimally invasive dentistry. Matches natural fluorescence of enamel. Good for Minimal Intervention (MI) treatment. Contains fluoride.

INDICATIONS

- Class V cavities
- Class III cavities
- Class I and II cavities in deciduous teeth
- Non-load bearing Class I and II cavities in permanent teeth
- Pits and fissures

CONTRA-INDICATIONS

Pulp capping

CONTENTS

50 capsules, instructions for use

PROCEDURE / PRECAUTIONS

CAVITY PREPARATION

Use minimal tooth reduction whenever possible. Calcium hydroxide liners need only to be used in deep cavities. For areas of uncut surfaces, to which adhesion is required, apply Toothcleanser (25% Polyacrylic acid solution) for a maximum of 30 seconds using a pledget of cotton wool.

Wash with water and dry with oil-free air. In areas, which are heavily coated with plaque or tartar, Prophylaxis with oil free Prophylaxepaste prior to tooth cleansing is required. If freshly cut dentine or enamel is contaminated with saliva, apply Toothcleanser for 10 seconds only before washing with water and drying with oil free air.

MIXING

1. Remove a capsule from an aluminium bag immediately before mixing. Tap gently on a hard surface to loosen any compacted powder.

2. Rotate the plunger clockwise until it stops securely (30°) (Fig. A).

• **Ensure you do not lift the nozzle before mixing.**

3. Press the plunger completely to ensure that all of the liquid is pressed into the powder (Fig. B)

4. Insert the capsule into the mixer and mix for 10 sec. (frequency: 4000 rpm or more). (Fig. C).

• **Important-** the capsule should not be pressed by the applicator before mixing.

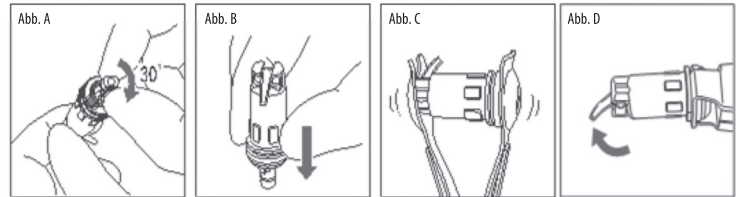
(3) APPLICATION

1. After mixing, remove the capsule from the mixer immediately. Then, place it into the applicator.

2. Lift the nozzle completely to dispense the paste (Fig. D).

• If the nozzle is not lifted completely, the paste cannot be dispensed correctly.

3. Dispense and apply the material to the restoration within 30 seconds after mixing.

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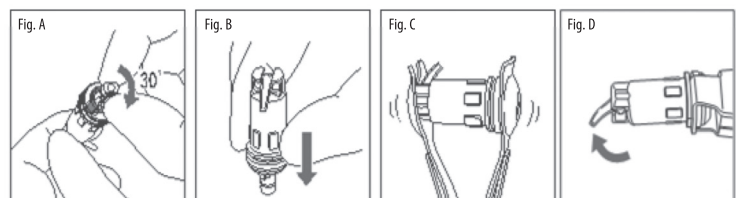
DO NOT LIFT THE NOZZLE BEFORE MIXING

FINISHING

Best results are obtained by polishing with abrasive discs or stones at a subsequent visit using water spray lubrication. It is possible to start the finishing and contouring process after 3 minutes from start of mixing but optimum aesthetics and hardness are achieved after 7 minutes from start of mixing. It is recommended to use petroleum jelly as a lubricant to prevent excess heat and desiccation of cement. For optimum polished surface white stones and alumina discs should be used. After finishing, it is recommended to coat the surface with glaze or varnish for protection from moisture.

SINGLE USE

⚠ The capsule may only be used once due to hygienic reasons (prevention of cross-contamination between patients).

WARNING

Do not expose patients or users known to be allergic to this type of material. Avoid contact of liquid and powder with oral mucosa, eyes, and skin. In case of contact, wash thoroughly with water and obtain medical advice.

STORAGE

☀️🌬️🌡️ Store in a cool, dry place (5-25°C).

BATCH CODE

☞ The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

☞ The expiry date is shown in year, month format. Do not use the product after this date.

For test purposes, the powder: liquid ratio is 1.30g to 0.50g at 23 ± 1°C. RH 50±-10%.

ISO 9917-1 net setting time: 2 minutes to 4 minutes from end of mix at 36-38°C.

Average contents per capsule: 0.27g Powder and 0.11g Liquid.

Minimum net volume of mixed cement per capsule: 0.144ml.

REF 400 102 – A2

REF 400 103 – A3

AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging.

Advanced Healthcare Ltd. Tonbridge, Kent, TN11 8JU, UK

EC REF Advena Ltd. · Tower Business Centre, 2nd Flr., Tower Street, Swatar, · BKR 4013 Malta.

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Müller & Weygandt GmbH · Reichardsweide 40 · 63654 Büdingen/Germnay · Tel.: +49 (0) 60 42 - 88 00 88