Adhesive for Identium® impression materials (Vinylsiloxanether™) ISO 4823

IDENTIUM® ADHESIVE

IDENTIUM® ADHESIVE

INSTRUCTIONS FOR USE / Gebrauchsinformation / Mode d'emploi / Gebruiksaanwijzing / Instruccija по использованию / Istruzioni per l'uso / Instrucciones de uso / Kullannin Klavuzu / Instruções de uso / Instrukcja użycia / Инструкция для застосування / 取扱説明書 / تعليمات الاستخدام

IDENTIUM® ADHESIVE

Haftrack für Identiu™ Abformmaterialien (Vinylsiloxanether™) ISO 4823
Adhesief voor Identiu™ impression materials (Vinylsiloxanether™) ISO 4823
Adhésif pour des matériaux d’empreinte Identiu™ (Vinylsiloxanether™) ISO 4823
Adhesief voor Identiu™ afdrukmaterialen (Vinylsiloxanether™) ISO 4823
Агезив для оттисковых материалов Identiu™ (винилсилоксанэфир (Vinylsiloxanether™)) согл. ISO 4823

Adhesivo per materiale da impronta Identiu™ (Vinylsiloxanether™) ISO 4823
Adhesivo para materiales impresión Identiu™ (Vinylsiloxanether™) ISO 4823
Identium™ ölçü materyalleri için adeziv (Vinylsiloxanether™) ISO 4823
Adesivo para materiais de impressão Identiu™ (Vinylsiloxanether™) ISO 4823
Klej do mas wyciskowych Identiu™ (Vinylsiloxanether™) ISO 4823
Identium™ (Vinylsiloxanether™) ISO 4823
Агезив для силиконов адитивной полимеризацiи компани Kettenbach
Identium™ 印章材用接着剂 (Vinylsiloxanether™) ISO 4823
4823
لازمه لاسوق مواد طبعات الاسنان™ Identiu™ (الفينيل اثير السيلكون) تبعا لتعليمات الشركة ايزو 4823

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ENGLISH

Product Description/Indication

Adhesive for Identiu™ impression materials (Vinylsiloxanether™). Adheres to metals and acrylics.

Instructions for use

Apply a thin coat of Identiu™ Adhesive onto the dry impression tray and allow it to dry for approx. 5 minutes.

Warmth and compressed air curtail the drying time. When using plastic impression trays, please check that they are compatible with the adhesive. Replace the cap on the bottle immediately after use!

After mixing the impression material, load it into the tray within 30 minutes.

Remains of impression material can be cleaned off the tray with cleaning solvent, petroleum ether or orange oil.

Warnings

Do not use Identiu™ Adhesive with condensation-curing silicones, addition-curing silicones, polyether or polysulfide materials.

Cautions

Do not use after expiration date.

Do not swallow the material! If accidentally swallowed: consult a medical doctor if problems arise or persist.

Avoid contact with the eyes. If there is accidental contact with the eyes: rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Avoid contact with skin and mucosa.

Do not allow the material to enter the sewer or water system to avoid environmental hazard.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

Keep away from sources of ignition.

Side Effects

Although there are no known systemic side effects, specific contact allergies cannot be ruled out in rare cases.

Do not use the material in case of known allergy to one of the ingredients.

Composition

Silicone resin, solvent (ethyl acetate, xylene)

Storage Conditions

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

Disposal

The empty bottles can be disposed in the household rubbish.

Trademarks

Identium™ Adhesive is registered trademark of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials are available in certain markets only.

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

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