Instructions

SUGGESTED PIN PLACEMENT PROCEDURE

1. Select pin sites. Keep in mind that MAX Restorative Pins shall not be bent.
2. Preparation of drilling site: With a rotary instrument create a flat surface approximately 2mm in diameter.
3. Create shallow indents at channel sites with 1/4 round bur in a high speed contra angle handpiece (**Fig. 1**).
4. With MAX Depth Limiting Drill in a slow speed contra angle handpiece (**Fig. 2**), prepare channel to a depth just short of the counter bore (**Fig. 3**).
5. Remove drill from channel while drill is still rotating (**Fig. 4**). Remove debris from drill and channel with air/water spray.
6. Complete channel preparation to full depth (colored drill shank acts as stop) (**Fig. 5**). The MAX Depth Limiting Drill automatically creates the correct counter bore for the MAX shoulder stop (**Fig. 6**).
7. As with all pin channels, channels should first be dried and cavity varnish applied with a Hygenic Paper Point.
8. Insert a MAX Pin into a latch type reduction gear handpiece (or into the Universal Hand Driver if placing manually). Position pin over channel before activating handpiece for insertion.
9. Operate handpiece at a very low speed and apply light downward pressure. The MAX Pin will align and thread into the prepared channel and shear when seated at the shoulder stop (**Fig. 7**).
10. Build up with restorative material of choice (**Fig. 8**).

NOTES AND PRECAUTIONS

- Channels should be at least 1.0mm from the dento-enamel junction and at least 4mm from each other.
- Rotate drill in a clockwise direction. The drill must be kept in a continuous clockwise rotation during the entire drilling procedure until the drill has been fully withdrawn from the channel.
- Do not bend Pins.
- Blunt drills may cause oversizing of channels. Always use sharp drills. Replace drills every 12-14 channels.
- *Max Depth Limiting Drills are precise delicate instruments. Used according to instructions they will allow you to prepare ideal channels for your MAX Restorative Pins. Improper use will result in premature drill failure and breakage.*

CLEANING, DISINFECTION AND STERILIZATION

Drills, TMS Bending Tool and Hand Wrench: Prior to each use, remove debris ultrasonically and sterilize by autoclave ONLY between patient use. (To minimize corrosion, place drills in separate bags from other instruments). Immediately air blow dry. A disinfectant may be used prior to use.

Pins: Prior to each use, disinfect or sterilize pins (autoclave ONLY).

Caution: Federal law restricts this device to sale by or on the order of a dentist

Coltène/Whaledent Inc.
235 Ascot Parkway
Cuyahoga Falls, OH 44223 / USA
Tel. USA & Canada 1 800 221 3046
+1 330 916 8800
Fax +1 330 916 7077
info@coltenewhaledent.com

Coltène/Whaledent AG
Feldwiesenstrasse 20
9450 Altstätten / Switzerland
Tel. +41 (0) 71 757 53 00
Fax +41 (0) 71 757 53 01
office@coltenewhaledent.ch

Authorized EU Representative:
Coltène/Whaledent GmbH + Co. KG
Raiffeisenstrasse 30
89129 Langenau / Germany
Tel. +49 (0) 7345 805 0
Fax +49 (0) 7345 805 201
productinfo@coltenewhaledent.de

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