Instructions for use

Arti-Spot® High-Spot Indicator

BK 85, BK 86, BK 87

Manufacturer

Dr. Jean Bausch GmbH & Co. KG
Oskar-Schindler-Str. 4
50769 Cologne
Germany

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Date of issue: 2020-03-27

General description and intended purpose of the medical product

Arti-Spot® is a contact color for testing the accurate fit of crowns, inlays, onlays, telescope crowns, clasps as well as friction surfaces of attachments and occlusal surfaces (**do not apply in oral cavity**), like gold or ceramics. Arti-Spot® can be applied to the surface to be tested with the enclosed brush (only for use in the laboratory) or with a disposable application brush, leaving a thin film of paint within a few seconds, when the solvent has evaporated.

The thin color skin is destroyed exactly at contact points and thus makes occlusion contacts or points of interferences visible when fitting crowns or telescopic crowns, for example.

For professional use only.

Indication

Arti-Spot® is a liquid contact color, available in the following colors: White (BK 85), Red (BK 86) and Blue (BK 87) for:

- Testing the accurate fit of crowns, inlays, onlays, telescope crowns, clasps as well as friction surfaces of attachments and occlusal surfaces (**do not apply in oral cavity**).

Contra indication

- Arti-Spot® contains solvents and therefore must not be applied directly invasively. The contact color must be applied outside the oral cavity on the surfaces of crowns, inlays, onlays, telescope crowns, clasps and frictions to be tested and these may only be applied to the oral cavity (invasively applied) after the color has dried (without solvent).

Side-effects

- Possible side-effects can be allergic reactions.
- Arti-Spot® can cause discoloration of prosthetic restorations made of plastic. It should only be used on metal or ceramic surfaces.

Notes

- The product must be stored in the sealed bottle at temperatures between 0°C and 40°C.
- The product contains a solvent that can evaporate quickly. After use, the bottle must be closed again immediately.
- The solvent contained in the product is highly flammable. Do not use the product near sources of ignition such as Bunsen burners (see also point Symbols).
- When used on patients, always apply the paint to the parts to be tested with a disposable brush outside the mouth.
- Arti-Spot® can be diluted with concentrated medical alcohol (ethanol 96%) to adjust the color viscosity.
- Arti-Spot® may cause discoloration of prosthetic restorations made of plastic. It should only be used on metal or ceramic surfaces.
- Arti-Spot® can be applied to Bausch Fleximeter-Strips® for combined usage.
- Stains on plastic surfaces or other materials can be removed with a dry paper towel. Color residues can also be removed with hot water or a steam jet. Tenacious paint residues can, if necessary, be dissolved and removed with concentrated alcohol (ethanol 96%).

Package content

- 1 bottle Arti-Spot®, contents 15ml (screw cap with holder for the brush)
- 1 brush
- Quick guide

Preparation

- Shake the bottle briefly before use (mixing balls are audible) so that the paint is mixed evenly; if the paint is too viscous, it can be diluted with medical alcohol (ethanol 96%, undenatured) at any time.

Instructions for use

Arti-Spot® High-Spot Indicator

BK 85, BK 86, BK 87

Application

- Arti-Spot® is used in accordance with normal practice hygiene.
- Color application (and start of application)
- Apply Arti-Spot® sparingly with a brush to the surface to be tested.
 - Let the paint dry for approx. 10 seconds.
 - Subsequently insert the part to be tested (crown, bridge, abutment of implants, etc.). The color skin is destroyed at the point of contact and the contact point is thus visible.
 - Do not reuse the application brush, to avoid transmission of germs!
 - After adjustment remove excessive color.

Effect/Diagnosis

After insertion of the surface to be tested into the oral cavity, interferences can be detected on the occlusal surfaces. The color skin is being destroyed on these occlusal surfaces. The base material then shines clearly through and highspots can easily be detected.

Reusability

- It is not permitted to reuse Single Use items (single use application brush). Safe use cannot be guaranteed if these products are used again, since there is an infection risk and / or the safety of the products is not liable.
- The use of damaged or soiled instruments is the responsibility of the user.
- In case of non-compliance, all liability is excluded.

Disposal instructions

Arti-Spot® can be disposed of with the usual practice waste.

Notification of incidents

Serious adverse events occurring in connection with the product must be reported to the manufacturer and the competent authority of the Member State.

Symbols

Manufacturer

Expiry date

„Follow manufacturer’s instructions“

Reference number

Batch code

Unique Device Identifier

Instructions for use
Arti-Spot® High-Spot Indicator
BK 85, BK 86, BK 87

Permissible storage temperature range

Conformity with the relevant EU regulations

H224 Extremely flammable liquid and vapor
P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources.
No smoking.
P403 Store in well-ventilated place.
P235 Keep cool.

H319 Causes serious eye irritation.
P280 Wear protective gloves/clothing and eye/face protection.
P305 IF IN EYES:
P351 Rinse cautiously with water for several minutes.
P338 Remove contact lenses, if present and easy to do. Continue rinsing.
P337 If eye irritation persists:
P313 Get medical advice/attention.
H336 May cause drowsiness or dizziness.
P261 Avoid breathing vapour or spray.
P233 Keep container tightly closed.

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