VOCO

Admira® Fusion x-base

In accordance with DIN EN ISO 4049

Instructions for use

Product description:

Admira Fusion x-base is a flowable, light-curing, radiopaque nano-hybrid ORMOCER® base restorative material. It is characterised by an increased depth of cure while at the same time showing reduced shrinkage stress, which means that it can be cured in layers of 4 mm thickness.

When used as a base/liner **Admira Fusion x-base** has to be overlaid occlusally with a covering layer, of at least 2 mm, of a universal or posterior composite/ORMOCER®. **Admira Fusion x-base** contains 72 % w/w inorganic fillers and is used with a dentine-enamel bond.

Shade:

Universal (U)

Indications:

- Base in class I and II cavities
- Cavity lining under direct restorative materials in class I and II cavities
- Small, non occlusal stress-bearing class I restorations according to minimally invasive filling therapy
- Class III and V restorations
- Extended fissure sealing
- Undercut blockout
- Repair of small enamel defects
- Repair of small defects in aesthetic indirect restorations
- Repair of temporary C&B-materials
- Core build-up

Preparation:

Clean the teeth to be treated. If required mark occlusal contact points. Allow the material to reach room temperature before use.

Cavity preparation:

In principle, cavity preparation should be carried out according to the rules of adhesive filling therapy and should be minimally invasive to conserve healthy tooth substance. Bevel the enamel margins on anteriors. Clean and dry the cavity afterwards. Non-carious cervical lesions do not have to be prepared; thorough cleaning is sufficient here.

Creating a dry working field:

Ensure that the work area is sufficiently dry. Use of a rubber dam is recommended.

Matrix placement:

Take appropriate measures for areas where the applied **Admira Fusion x-base** can flow away. Translucent matrices that are wedged in the approximal area are advantageous. Minimal separation facilitates the shaping of the approximal contact and placement of the matrix.

Pulp protection:

A suitable pulp-protective liner that can be covered with a stable cement, if necessary, should be applied in close proximity to the pulp.

Bonding material:

Admira Fusion x-base is used in the adhesive technique with a dentine-enamel bond. Any light-curing bond material can be used. Follow the respective instructions for use with regard to preparation (etching technique) and application.

Application of Admira Fusion x-base:

Lock the enclosed application cannule on the syringe with a clockwise turn and directly apply **Admira Fusion x-base**. Do not pull the plunger back on the syringe during or after use to guarantee the functioning of the non-dripping syringe. Begin applying the material directly into the prepared cavity at the deepest point and fill the cavity from bottom to top. Keep the tip of the metal cannula immersed all times. To prevent the formation of air bubbles ensure a slow, even flow. If necessary, air bubbles or excess material can be removed before polymerisation. When used as a base/liner the distance to the occlusal margin of the cavity must be at least 2 mm. **Admira Fusion x-base** can be polymerised in increments of up to 4 mm.

Light-curing:

Conventional polymerisation devices are suitable for light-curing the material. When using an LED-/halogen light with a minimum light output of 800 mW/cm² the curing time is 20 s, for a minimum light output of 500 mW/cm² the curing time is 40 s.

Hold the light emission tip of the device as close as possible to the surface of the filling. Otherwise, the curing depth may be reduced. Incomplete curing may lead to discoloration and discomfort.

Final occlusal layer:

When used as a base/liner **Admira Fusion x-base** has to be overlaid with a covering layer, of at least 2 mm, of a universal or posterior composite/ORMOCER®. For working and finishing directions follow the manufacturer's instructions for use.

Finishing:

The restoration can be finished and polished immediately after removing the matrices (e.g., fine or extra-fine diamond bur, polisher). The tooth should be fluoridated as a final step.

Information, precautionary measures:

- **Admira Fusion x-base** contains methacrylates and BHT. **Admira Fusion x-base** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.
- The enclosed application cannulae are intended for single use and must be disposed of after only one use. Return the cap to the syringe after use.
- Phenolic substances, especially preparations containing eugenol or thymol, interfere with the curing of filling composites. Avoid the use of zinc oxide eugenol cements or other eugenol-containing materials in combination with filling composites.

Storage:

Store at temperatures of 4 °C to 23 °C. If refrigerated, the material must be allowed to reach room temperature before use. To avoid exposure to light and potentially resulting polymerisation, syringes must be closed immediately after dispensing. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA2812 E1 0117 99 © by VOCO
ORMOCER® is a registered trademark of Fraunhofer Gesellschaft zur Förderung der angewandten Forschung e.V., München, Germany.
VOCO GmbH