# PROPHY PERIO

Stand:04/2019

**REF** 100 891 **Inhalt/Content:** e 120 g

Ⓒ PROPHY PERIO

Ⓕ PROPHY PERIO

Ⓘ PROPHY PERIO

Ⓟ PROPHY PERIO

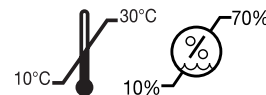
Ⓒ PROPHY PERIO

Ⓗ PROPHY PERIO

**LOT**

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## Ⓧ PROPHY PERIO

### Type of application

Prophylaxis powder based on natural sugars (non-cariogenic) for the gentle removal of supra and subgingival deposits and discolouration. Suitable for all commercially

available powder blasting devices.

### Notes on application

Please observe the instructions for use and the maximum filling level of the device manufacturers.

### Subgingival application

- Removal of periodontal biofilms.
- Cleaning of implants
- For supportive periodontal therapy (SPT) following completion of initial treatment

### Supragingival application

- Removal of supragingival biofilms and discolouration around the sulcus
- Removal of discolouration in sensitive areas.
- Polishing of teeth, a final polish is superfluous
- Suitable for regular prophylaxis treatment

Patients suffering from chronic bronchitis or asthma must never be treated with a powder jet device. Patient and operator must wear eye protection. The operator must wear a mask and gloves. Use a high performance suction device. Apply petroleum jelly to the patient's lips. The powder jet must not be directed at fillings, crowns and bridges; these restorations could be damaged. For dental use only. Keep container closed. **Shake thoroughly before use!**

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