GI-MASK®Automix

New Formula

Gebrauchsinformation DE

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12-2018

Instructions for use EN

Definition
GI-MASK Automix New Formula is a medium viscosity, thixotropic and dimensionally stable dental impression material, based on an A-silicone for use in dentistry with an automatic mixing device, for the construction of gingival masks in dental laboratories.

Material type
Polyvinylsilicone, addition type, silicone elastomer.
Category: pink
Catalyst: white
ISO 4823, Typ 2, medium body consistency.

Indications

- Fast and accurate reconstruction of the complete contour of the gingiva around the preparation on the model. It is particularly suitable for single crowns, bridges, implants, combination and telescopic work.
- For implant techniques, the special viscosity allows the direct application of the model analog to the silicone. The production of a Lab Putty key is not necessary.
- For polyether impression techniques: application only on the described indirect technique. For direct technique: use Coltene G-Mask-C-Silicone, art. no. 8030.

Contra-indications

GI-MASK Automix New Formula should not be used for the fabrication of epitheses.

Effects secondary

Polysilvinilicone have a very good biological compatibility and no harmful reactions or secondary effects are known.

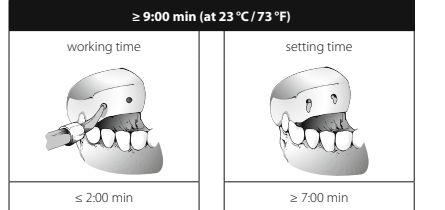
Interactions

The secretions cutaneous, the gants in latex and the surfaces contaminated by the gants in latex may influence the process of curing of polysilvinilicone. The same applies to preparations containing eugenol or other substances that affect the curing process.

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Clinical time

Contra-indications

This material can not be used for epitheses.

Side effects

Polyvinylsilicones have very good biological compatibility and no harmful reactions or secondary effects are known.

Interactions

skin secretion, latex gloves and surfaces contaminated by these gloves (teeth, preparations, extraction cords, etc.) may interfere with the setting process of polyvinylsilicones. Compounds containing eugenol or hemostatic substances can also impede perfect setting.

Mixing and application of GI-MASK Automix New Formula

- Insert the cartridge filled with GI-MASK Automix New Formula into the opened dispenser of the 50/75 ml COLTENE Dispenser. Close the release plate.
- Remove cartridge cap.
- Squeeze the trigger slowly and extrude a small quantity of base and catalyst on a paper towel until it is evident that both are flowing evenly.
- Remove any contamination with water.
- Set the mixing tip onto the cartridge and twist a ¼ turn clockwise to lock it in place.
- Place applicator tip on mixing tip.
- Eject material by pulling the trigger.
- After using the cartridge don't remove the mixing tip. This seals the cartridge until it is used again and prevents cross contamination.
- Only remove the mixing tip directly before the device to be used again.

Application 1: Indirect technique

- Build a mould by creating a crank around all parts of the model to be masked, using the material of the COLTENE Lab- Putty.
- Grind away a layer of material in those areas to be modelled in GI-MASK Automix New Formula.
- After the injection holes in the silicone mould from the palatal/lingual and lingual side. To avoid the gingiva mask also fill air vents into the prepared area.
- On the buccal/labial side. It is important to place the holes on mould around.
- On the buccal/labial side. It is important to place the holes on mould around.
- Afterwards working with the material of the COLTENE Lab-Putty evenly with GI-MASK Universal Separator and air dry for at least 5 minutes.
- Afterwards working on the sectioned and trimmed model.
- Hold material securely in place. Start injecting the material with mixing and application tip into the first hole (ie. palatal/lingual side) until the COLTENE G-Mask Automix New Formula is extruded from the hole on the opposite side (buccal/labial side).
- GI-MASK Automix New Formula will set in approximately 9 minutes, from the start of the mix. As soon as the GI-MASK Automix New Formula has set, carefully remove the matrix from the model.
- Cut the cutters in the area of the ink and outlet ducts and lift the GI-MASK Automix New Formula from the matrix using a scalpel.
- Remove any excess material with fine sieves, a surgical knife or a rotary instrument for silicones (ie. Trimmer for silicones, Art.No. 8075).
- The gingival mask can now be removed safely from the model and repositioned.

Application 2: Direct technique

- Remove the impression with running, luke-warm water, clean and dry the model.
- Apply spray the material with pressure using GI-MASK Automix Separator and air dry for at least 5 minutes. Inject GI-MASK Automix New Formula directly in the impression.
- Springe around the model analog (implantology) up to 5 mm by using the application tip.
- Wait until GI-MASK Automix New Formula has set in about 9 minutes, from the start of the mix.
- Use the impression with plaster.
- Remove the material from the model.
- Remove any excess material with fine sieves, a surgical knife or a rotary instrument for silicones (ie. Trimmer for silicones, Art.No. 8075).
- The gingival mask can now be removed safely from the model and repositioned.

Shelf life and storage

GI-MASK Automix New Formula fulfills the intended purpose at least until the expiry date which is stored at room temperature, avoiding exposure to direct sunlight or other heat sources.

Marking

The expiry date and ISO number are shown on the package and cartridge.

Technical data ISO 4823

Temperature range: 23 °C/73 °F room temperature and 50% relative humidity. (Higher temperatures shorten and low temperatures lengthen the times)

Mixing time: 0,15 min
Total working time: 3,00 min
Recovery after deformation: 99,8 %
Dimensional change: 5,1 %

Applied to dentists and dental laboratories or upon their instructions.

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