M+W select MAXI WIPES

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☑ Type of application:
Dry fleece wipes for impregnating with suitable disinfectants. A ready-to-use disinfectant solution tested according to EN 16615 should be used to impregnate the wipes.
Impregnation:
Place the dry roll of wipes in the tightly closing dispensing tub. Then pour (Filling quantity of liquid: 1,9l M+W Spray disinfectant AF/Plus 1,7l M+W Spray disinfectant alcohol-free) disinfectant solution evenly over the roll from above.
Pull the first wipe of the fleece roll and thread through the cover closure. Then press the cover firmly on the tub and always keep closed tightly. Leave the soaked roll to stand for approx. 30 minutes to allow for uniform moistening. The moist wipes should not be used for longer than 28 days (service life).
Observe the impregnation date, the service life and other legally relevant information of the respective impregnating liquid and enter them in the corresponding area of the label. Then tear off the label and stick it on the dispensing tub. The dispensing tub is to be prepared according to the preparation instructions below with a suitable disinfection wipe.

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☑ M+W Maxi Wipes impregnated with M+W spray disinfectant AF/Plus

Type of application:
According to MPG: alcohol-based moist wipes for prophylactic surface disinfection of alcohol-resistant medical devices. Not suitable for the final (terminal) disinfection of invasive medical devices

According to the Biocide Directive:
alcohol-based moist wipes for fast disinfection of alcohol-resistant surfaces.

Safety notes:
H 228 Flammable solid. Keep away from heat/sparks/open flames/hot surfaces. No smoking. Wear protective gloves/eye protection. Store in a closed container. Flash point acc. to DIN 51755: approx. 25°C. Application max. 50 ml per m². The total amount applied per room must not exceed 100 ml per m² of floor area. Keep container tightly closed.
Only pass completely emptied containers to recycling. Dispose of residual material acc. to EAK 07 06 99 Directive 648/2004/EC (impregnating liquid): Contains: < 5% cationic surfactants, fragrances

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☑ M+W Maxi Wipes impregnated with M+W spray disinfectant, alcohol-free

Type of application:
According to MPG: alcohol-free moist wipes for prophylactic surface disinfection of medical devices. Not suitable for the final (terminal) disinfection of invasive medical devices.

According to the Biocide Directive:
alcohol-free disinfection wipes for surface desinfection

Notes on application:
Before using on large surfaces with sensitive materials (for example, artificial leather), test first for material compatibility on a small section. To avoid streaking, the treated surfaces should be wiped at regular intervals with water. Wear suitable gloves during application (for example, made of nitrile rubber).
Only pass completely emptied containers to recycling. Dispose of residual material acc. to EAK 07 06 99 Directive 648/2004/EC (impregnating liquid): Contains: < 5% cationic surfactants, fragrances

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