# TRAY CLEANER

**Inhalt/Contents/**

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**Obsah: e 1 Liter/ltr./**

**litre/litr/litr**

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Ⓔ The tray cleaner is a concentrate for the careful cleaning of shaping spoons and instruments. The special cleaner quickly removes residues of alginates, zinc oxide eugenol, carboxylate- and phosphate cements and thermoplastic moulding compounds and waxes. The constituents are bio-degradable.

**Application:** The tray cleaner is used at a concentration of 5-10%, and in the case of heavy soiling can be used as a pure concentrate. Add approx. 1 full dosing chamber per ltr. of warm water. The material residues are softened by the solution within a few hours, and can then be removed easily with a brush. The solution can also be used in an ultrasonic bath. The special cleaner is suitable stainless items. Chrome, nickel and aluminium parts should not be cleaned together. Danger of corrosion!

**H318** Causes serious eye damage.

**H315** Causes skin irritation.

**H226** Flammable liquid and vapour.

Keep away from heat/sparks/open flames/hot surfaces. - No smoking.

Keep container tightly closed.

Wear protective gloves and eye protection.

IF ON SKIN: Wash with plenty of soap and water.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Immediately call a POISON CENTER or doctor/physician.

Call a POISON CENTER or doctor/physician if you feel unwell.

If skin irritation occurs: Get medical advice / attention.

In case of fire: Use for extinction: CO2, powder or water spray.

or water spray.

In case of fire: Use for extinction: CO2, powder or water spray.

Store in a well-ventilated place. Keep cool.

Dispose of contents/container in accordance with local/ regional/national/international regulations.

Ingredients:

648/2004/EC: Between 5% und 15% anionic Tenside, between 15% and 30% Na-NTA, Scent.

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