WET WIPES SOFT ALKOHLFREI

Stand: 11/04/18

REF 100 758

Inhalt/Content/Contenu/Zawartość/Obşah:
90 Stück/pcs/unités/sztuk/ks

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GB Alcohol-free moist wipes for surface disinfection of medical devices.

Type of application

Alcohol-free moist wipes for prophylactic surface disinfection of non-invasive medical devices. Particularly suited for use on alcohol-sensitive surfaces such as acrylic glass.

Notes on application:

Remove the stick & lift label from the bag, stick it to the intended dispensing tub and add the use-by date etc. Open the bag at the top and place in the dispensing tub. Pull the first wipe and thread it through the star in the cover closure. Press the cover firmly on the container and close tightly.

After opening the bag, the wipes can be used for 28 days in the dispensing container, always close the container.

Before using on large surfaces with sensitive materials, for example, artificial leather, test first for material compatibility on a small hidden section. To avoid streaking, the treated surfaces should be wiped at regular intervals with a moist (water) disposable cloth. The dispensing tub is to be prepared according to the preparation instructions opposite with a suitable disinfection wipe. Wear suitable gloves during application.

Spectrum of activity:

Bactericidal (incl. MRSA) acc. to VAH, EN 13727, EN 13697, EN 1276; EN 16615, levurocidal (Candida albicans) acc. to VAH, EN 13624, EN 13697, EN 1650; EN 16615, "limited virucidal" (effective against enveloped viruses, i.e. HBV, HIV, HCV), rotavirus acc. to RKI/DVV, norovirus acc. to EN 14476.

Exposure times (in relation to the impregnating liquid):

Bacteria according to EN 13727 and EN 13697 (high load):	Conc. 1 min
Candida albicans according to EN 13624 and EN 13697 (high load):	Conc. 1 min
Enveloped virus (i.e. HBV, HIV, HCV) acc. to RKI/DVV with high load:	Conc. 1 min
Rotavirus acc. to RKI/DVV with load:	Conc. 1 min
Norovirus acc. to EN 14476 (low load):	Conc. 30 min
Efficacy of the wet wipes acc. to EN 16615 and VAH (high load):	Conc. 1 min

Bag opened on: _____

Use-by date: _____ (= date of opening plus 28 days)

Signature: _____

Bacteria and Candida albicans (impregnating liquid, included in VAH listing):

Conc. 5 minutes (shortest exposure time for surface disinfectants in VAH list)

Content:

90 pieces made of PES/VIS material mix; 28 x 30 cm impregnated

100 g of impregnating liquid contain:

0.24g benzyl-C12-16-alkyldimethyl ammonium chloride

0.24g N,N-didecyl-N,N-dimethyl ammonium chloride

0.24g C12-14-alkyl[(ethylphenyl)methyl] dimethyl ammonium chloride

Only pass completely emptied containers to recycling. Dispose of residual material acc. to EAK 07 06 99.

648/2005/EC: Directive 648/2004/EC: Contains <5% cationic surfactants, fragrances.

1 Processing of dispenser box
Remove lid from container

2 Wipe lid and inside of
container, the wipe tear-
off and reservoir with an
impregnated disinfection wipe
(observe exposure time of
disinfectant).

3 Allow container and
lid to dry

4 Cut the filled bag open at the
top and place in the container.
The disinfection wipes are
ready for use.

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