MUNDSPÜLBECKEN- REINIGER AF

REF 100 732

Inhalt/Content/Contenu/Zawartość/Obsah/Tartalma:

e 750 ml

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☞ Type of application:

The M+W Mundspülbeckenreiniger AF is a special alkaline preparation for cleaning, disinfection and care of mouth rinse basins in dental treatment units. M+W Mundspülbeckenreiniger AF reliably removes deposits of scale, blood and colouring agent residues from plaque tablets.

Notes on application:

M+W Mundspülbeckenreiniger AF is applied undiluted. To open, press the safety closure together on the smooth surfaces and twist it. Moisten the mouth rinse basin with approx. 20ml of the cleanser and let it work in for a short time or apply with a sponge or a brush. Rinse off with water after at least one minute of exposure time. Use is recommended after each contamination.

Exposure times:

Bacteria and *Candida albicans* according to EN 13697 and DGHM (with mechanics, low load): Konz. 1 min.
TbB (M.terrae) according to EN 14348 (low load): Konz. 1 min.

Safety notes:

H226 Flammable liquid and vapour. **H314** Causes severe skin burns and eye damage. **H412** Harmful to aquatic life with long lasting effects. Keep away from heat, hot surfaces, sparks, open flames and other sources of ignition. No smoking. Do not inhale

mist/ vapour/ aerosol. Wear protective gloves/eye protection. Avoid release to the environment. IF ON SKIN: Remove all contaminated clothing immediately. Rinse skin with water/shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTRE/doctor. IF SWALLOWED: Rinse mouth. DO NOT induce vomiting. Store in a well-ventilated place. Keep in a cool place. Disposal of contents / container in accordance with local / regional / national / international regulations.

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