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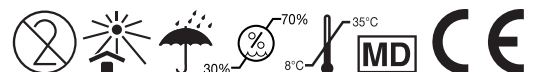
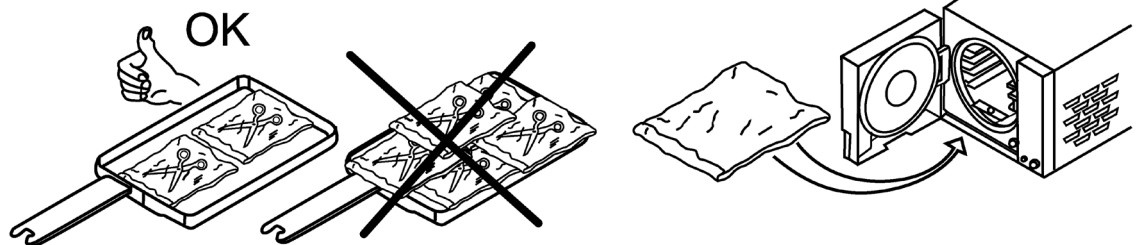
# KOMBIFOLIE

## Ⓜ ADVICES FOR A CORRECT USE

In order to guarantee optimal performances of the sterilization rolls and to reduce to the minimum the risk of the opening of the weldings, we recommend to scrupulously observe the following indications.

### In general

- Avoid overloading the chamber of the autoclave over the allowed weight (see autoclave instructions manual).
- Place the objects sufficiently distant between them and do not stack them so that they remain separated for the entire sterilization cycle.
- Leave space between one tray and another in order to allow the circulation of the vapour thus helping the drying.

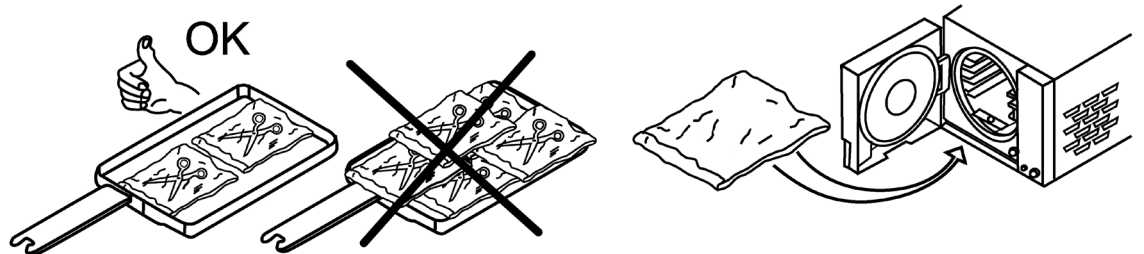
### Wrapped objects

- Place the pouch according to the specifications of the autoclave.
- Do not stack the pouches on the trays.

- The objects to be wrapped must be placed in an orderly way, at the centre of the pouch, leaving about 1 cm of space between the objects and the weldings on the sides.
- Said objects must be placed furthermore lengthwise with respect to the lateral weldings of the rolls, avoiding that any part pushes on the weldings themselves.
- If these limits cannot be respected, use a larger sterilization pouch or roll. Please remember that the sterilization rolls are available with the following widths in mm: 50 – 75 – 100 – 200.

The quality of our products are of the maximum levels because they are produced with the best technology and best materials available today. Before being sold, they are always randomly tested with sterilization tests, which simulate the typical conditions of their use.

They conform to the Directives 93/42/EEC and 2007/47/EC and to the Standards UNI EN 868-5 and EN ISO 11607.

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