# PROVISO

## D Provisorischer Füllzement

### Produktbeschreibung

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## GB Temporary filling cement

### Definition

Zinc oxide and synthetic materials temporary filling cement, self hardening, ready to use. No expansion when setting, due to its high quality synthetic materials. Eugenol-free to not hinder polymerization of resins and composites. Contains fluoride for a cariostatic action.

### Safety notes

#### Warnings

- M+W Proviso contains Zinc Sulfate which may seriously irritate eyes.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek for medical advice.

### Precautions

- Wear suitable protective eyewear, mask, clothing and gloves.

### Contraindication

M+W Proviso is contraindicated in patients with a history of allergic reaction to Zinc Sulfate or any other of the components.

### Instructions for use

Remove the desired amount of cement with a spatula. Tightly close the jar after use.

### Application

Apply the cement in the cavity, with a spatula, hardening start immediately with saliva. Remove it with an ultrasonic device or with a probe.

### Storage and shelf-life

Store at a temperature between 12°C and 25°C. When stored under such conditions, M+W Proviso has a shelf-life of 48 months. Store in a dry, cool and well-ventilated place, away from humidity.

### Presentations and references

#### 3 consistencies:

- #100 161 - glas, 40g red (hard)
- #100 160 - glas, 40g white (normal)
- #100 162 - glas, 40g grey (soft)

### Important notice

- Do not throw away the box and the instruction for use until the product is completely finished.
- High humidity could damage the packaging on which important instructions a legal information are printed.
- Storage at a too high temperature could cause to age prematurely and soften the product.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

### Disclaimer

The materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the instructions for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the instructions. Descriptions and date constitute no warranty of attributes and are not binding.

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Vertrieb durch Müller & Weygandt GmbH · Reichardsweide 40 · D-63654 Büdingen · Tel.: +49 (0) 60 42 – 88 00 88