VOCO**Aqua Ionofil Plus****(GB)****CE** 0482

In accordance with DIN EN ISO 29917

Instructions for use**Product description:**

Aqua Ionofil Plus is a radiopaque, water-mixable glass ionomer filling cement with chemical adhesion to enamel and dentine. **Aqua Ionofil Plus** is available in shades A1, A2 and A3 and has enamel-like translucency.

Indications:

- Restoration of deciduous teeth (especially class I)
- Repair of V-shaped defects and enamel as well as root erosions in the dental neck area including class V fillings
- Fillings for cavities caused by carious lesions
- Fillings of class III and small fillings of class I (incl. extended fissure sealing) as well as core build-up under crowns and as a cavity lining

Method of Use:

The adhesion of **Aqua Ionofil Plus** to enamel and dentine prevents the formation of marginal leakage and creates retentions. Undercuts provide additional hold to the filling. Layer thickness of the filling should not be less than 1 mm.

Clean the cavity (e.g. with a 3 % hydrogen peroxide), rinse carefully with water and dry. In case of a remaining dentine thickness of more than 1 - 1.5 mm no additional lining is necessary. In cases of lesser dentine thickness it is recommended to cover the areas in proximity of the pulp with calcium hydroxide.

Attention! Mix Aqua Ionofil Plus only with water.**Shake powder well before use.**

Allow **Aqua Ionofil Plus** to reach room temperature (max. 23 °C) before application. In case of higher temperatures it is recommended to use a cooled glass plate.

Dispense **Aqua Ionofil Plus** powder onto a glass plate or suitable mixing pad and mix the powder portion by portion with the water. Use 1 drop of water for 1 level measuring scoop of powder (mixing ratio powder : water = 5.4 - 6.3 g : 1).

When dispensing the water, hold the dropper bottle in an absolutely vertical position, making sure that the metal cannula protrudes approx. 2 - 3 mm from the dropper. Do not use drops for mixing which escape when tilting the bottle.

Use a strong plastic spatula or another abrasion-resistant spatula for mixing.

Mixing time	Working time	Setting time
30 - 40 s	2.5 - 3.5 min	5 - 6 min

The mixing time is approx. 30 - 40 s, but should not exceed 1 min.

Working time from end of mixing procedure is approx. 2 - 3 min (at room temperature). After the filling has been inserted into the cavity, the setting time of **Aqua Ionofil Plus** is approx. 2 - 3 min.

Ensure that the cavity remains dry during the filling procedure.

Class V cavities should be at least 1 mm in depth. Prevent feather-shaped margins. In case of doubt prepare in a conventional way.

During the setting phase, especially immediately after application, glass ionomer cements are sensitive to humidity (breath, saliva) and also to drying-out. Therefore, if possible, cover immediately with a strip or matrix or apply a protective varnish (see instructions for use **Final Varnish LC**).

The rough excess material can be removed with a sharp instrument after **Aqua Ionofil Plus** has set (no earlier than 4 - 6 min after application depending on mixing temperature and mixing ratio). Finish with an ultra-fine-grained diamond or a flexible disk. Afterwards, apply a coating of varnish. For best results, perform final polishing 24 h after placing the filling.

Precautions:

In order to prevent irritations in the proximity of the pulp (closer than 1 - 1.5 mm) use pulp protection. Tightly close powder after use to protect from moisture and loss of consistency. Avoid contact with the mucosa and eyes.

Aqua Ionofil Plus contains fluorosilicic glass and polyacrylic acid. **Aqua Ionofil Plus** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.

Do not use insulating agents on dentine in order to maintain maximum adhesion (with the exception of pulp protection).

Storage:

Store at temperatures between 4 °C - 23 °C.

Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA1509 E1 0114 99 © by VOCO
VOCO GmbH