# ÄTZGEL 36 %

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# ETCHING GEL 36%

Information and  
instructions for use

Stand:09/17

M+W etching gel is a 36% ortho-phosphoric acid etching gel with a perfect thixotropic flow behaviour. It is used for the etching of tooth substance as part of the melt-etch technique and the total-etch technique. If used in accordance with the safety and usage instructions, the product has no known side effects. Underfilling materials may be etched by the etch gel in case of longer acting times.

M+W etching gel 36% can be applied safely. Does not dry out and has a shelf-life of at least 2 years from the date of manufacture at a maximum storage temperature of 25°C.

- For dental use only!
- Do not use after the expiry date.
- Seal syringe immediately after use.

### Application:

Take applicator syringe from the pack and remove the cover cap.

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### Preparation

The areas of the tooth substance to be etched must be clearly defined.

- 1.) The use of a dental dam is recommended! Avoid all contact with soft tissue (skin, oral mucosa, eyes etc.). If the etch-gel comes into contact with soft tissue, rinse immediately with plenty of water and consult a doctor if necessary.
- 2.) Before preparation, clean the tooth using a rubber cup with fluoride- and fat-free polishing paste.
- 3.) Remove cleaning residues carefully with water and clean and dry the tooth surface with oil-free compressed air.

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4.) Prepare the cavity accordingly, chamfer the melt edges in order to produce permanent filling edges. Rinse out the cavity thoroughly with water and dry with oil-free compressed air. Avoid excessive drying!

5.) Protect the pulp in deep cavities (e.g. cover with a thin coating of a suitable underfilling material). The cavity should be dry and free of impurities, and excessive drying of the dentine should be avoided.

#### Etching process

6.) Apply the etching-gel to the melt edges of the cavity and if necessary to the dentine.

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15 - 30 sec. on dental enamel

10 - 15 sec. on dentine

50 - 80 sec. on deciduous teeth and teeth with a high fluoride content.

7.) Rinse for at least 20 sec. with water.

8.) Dry with oil- and water-free compressed air. The etched surface should not be touched or contaminated by saliva until continuation of the treatment. If contamination occurs after etching and drying, etching must be repeated briefly (approx. 10 sec.).

The etched enamel should appear chalky white!

After etching, the use of adhesives etc. can be continued in accordance with the manufacturer's usage information.

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#### Safety instructions

H 314 Causes severe skin burns and severe eye damage. Wear protective gloves / eye protection. **IF IN EYES:** Rinse carefully with water for several minutes. Remove contact lenses if possible. Continue rinsing. Call the **POISON INFORMATION CENTRE** or a doctor immediately. **IF SWALLOWED:** rinse mouth. **DO NOT** induce vomiting. **IF IN CONTACT WITH SKIN:** wash with copious amounts of water and soap. Store in a closed container. Disposal of contents / container in accordance with local / regional / national/international regulations. Contains ortho phosphoric acid.

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