SEPTOPRINT PLUS


REF 200 725 Inhalt/Content/Contenu/Contenuto/Ob sah/Tartalom: e 5 L

- Ⓓ Zur desinfizierenden Reinigung aller dentalen Abformmaterialien.
- ⒼⒷ For a disinfecting cleaning of all dental impression materials.
- Ⓕ Pour le nettoyage et la désinfection de tous les matériaux d'empreinte dentaire.
- Ⓘ Per la pulizia disinfettante di tutti i materiali per impronta dentale.
- ⒸⒶ K dezinfekčnímu čištění všech dentálních materiálů zubních otisků.
- Ⓗ Az összes fogászati lenyomatanyag fertőtlenítéssel jelleget tisztítására.

UFI: 4KX2-KA80-400N-07XT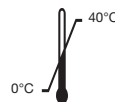

2024-03

PRISMAN GmbH · Otto-Hahn-Ring 6-18 · 64653 Lorsch, Germany · Tel.: +49 (0) 62 51-86 69 80-0

MW Dental

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Ⓔ Type of application:

Preparation for a disinfecting cleaning of all dental impression materials in an immersion bath procedure.

Notes on application:

Immerse the respective impression into the ready-to-use disinfectant solution, and remove again after the recommended contact time. The impression must be fully covered by the solution. Immerse alginates and hydrocolloids for a short time only and then let them dry. Afterwards, rinse the impression with water. Do not mix with cleaning agents or other disinfectants. Depending on the degree of contamination, the working solution may need to be replaced (RKI recommendation: at least once per working day). The solution can be disposed of via the wastewater. For professional users only. Suitable for use in dental practices and hospitals.

Contact times:

Bacteria (incl. MRSA) as per VAH, EN 13727 and EN 14561 (dirty conditions):	conc. 5 min.
Candida albicans as per VAH, EN 13624 and EN 14562 (dirty conditions):	conc. 5 min.
“Limited virucidal” (enveloped viruses e.g. HBV, HIV, HCV) as per EN 14476	
and EN 17111 (clean conditions):	conc. 5 min.
TB (M. terrae in a suspension test, clean conditions):	conc. 15 min.

Safety instructions:

H 412 Harmful to aquatic life with long lasting effects. Avoid release to the environment. Dispose of contents/container in accordance with all local, regional, national and international regulations.

All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority of the State in which the user/patient is established.

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