PERMABOND LAB NEW FORMULA

Stand: 210318

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Instructions for use

M+W Permabond Lab New Formula is an adhesion promoter for bonding a non-precious-metal or precious-metal alloy and a veneering composite.

Preparing the surface for veneering: Sandblast the alloy with 110 µm aluminium oxide at 3 bar pressure. Dry and clean the surface thoroughly with clean compressed air.

Application: Apply M+W Permabond Lab New Formula to the veneering surface in one coat with a brush and allow to set for 10 seconds. Gently disperse the volatile constituents with compressed air. The veneering composite can be applied to the pretreated surface in accordance with the instructions for use.

Warning: M+W Permabond Lab New Formula contains acetone. Acetone is volatile. Use only in well-ventilated areas. Avoid contact with eyes and mucous membranes. After contact with eyes rinse with copious amounts of water and consult an ophthalmologist. After contact with skin rinse with copious amounts of water.

Storage instructions: Do not store above 25°C. Do not use after the expiration date. **The product is volatile. Close container immediately after use.**

Guarantee: Our instructions for use, whether given orally or in writing, are based on our experience and can therefore only be considered as reference values. Our products are subject to continuous development. We reserve the right to make changes.

Side effects: Unwanted side effects of this medical product can be expected very rarely when processed and used as directed. However, immune reactions (e.g. allergies) or local effects cannot be completely excluded. If you become aware of unwanted side effects, even in cases of doubt, please inform us.

Contraindications / side effects: This product must not be used with hypersensitive patients or only under close supervision of a doctor or dentist. In such cases the composition of medical products that we supply is available on request. Known cross-reactions or interactions of the medical product with other substances in the mouth must be considered by the dentist when using the product.

Note: Please give all the above information to the dentist if you are using this medical product for a special fabrication. Note the material safety data sheets during processing.

Warranty: P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior Dental Materials GmbH product.

Limitation of liability: Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep out of reach of children.

For use by dentists and dental technicians only.

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Manufacturer: P.L. Superior Dental Materials GmbH · 22607 Hamburg · Germany