VOCO**Profibase****GB**

Instructions for use

CE**Product description:**

Profibase is a light-curing resin in pre-contoured plates for maxillae and mandibles. The material is ready for use and easy to handle.

Profibase can be processed under room and day light for a long time and cures quickly under halogen and UVA light. Even after curing, **Profibase** maintains its pleasant peppermint flavour.

Indications:

- Fabrication of individual impression and functional trays for use in crown, bridge and total prosthetic technique to be used for the fabrication of bases of bite patterns as well as for dimensionally stable base plates for wax contouring in the total prosthetic technique.

Method of use:**1. Fabrication of an individual tray**

Allow **Profibase** to reach room temperature before use. Mark margins for the individual tray on the plaster model. If necessary, use a space maintainer. If space maintainer is not necessary, insulate plaster model. Remove protective foil, mold plate onto the model and cut the desired rim as marked. A handle can easily be shaped from the excess material.

Profibase is then polymerised on the model with a standard light unit (halogen or UVA light). Depending on the thickness of the tray, polymerise the reverse side for 3 - 5 min. A longer polymerisation time improves the physical properties of the material. Therefore, prolong time of polymerisation if necessary. The individual tray is immediately ready for use and can be worked upon without delay. A slight inhibition layer, eventually caused by oxygen, can be easily removed with a tissue moistened with alcohol.

2. Fabrication of a base plate

Mark the margin of the planned prosthesis base on the plaster model. Block out undercuts as usual. Isolate the model thinly with an alginate isolation.

Carefully place a plate of **Profibase** onto the model, adjust the rims with a penknife and polymerise the model with base plate for 5 min in a standard halogen or UVA light unit. One single exposure is sufficient.

Supplementary information:

For finishing the rims and the ridge, hard metal burs are most suited.

Create cross-shaped retentions on the ridge before applying wax. Then a thin layer of sticky wax can be applied.

Precautionary measures/indications:

Profibase contains Bis-GMA, diurethane-dimethacrylate, TEGDMA, BHT, peppermint oil. **Profibase** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.

Insure sufficient and complete polymerisation! A too short exposure time or low intensity of light may lead to a loss of stability.

Storage:

Store at temperatures of 4 °C to 23 °C. Avoid direct sunlight. Excess material can be used again if stored under light-protection. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA2111 E1 0116 99 © by VOCO
VOCO GmbH