<p>CanalPro EDTA (DE)</p>

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<p>***</p>

<p>CanalPro EDTA (EN)</p>

Description:
EDTA solution 17% (ethylenediaminetetraacetic acid) is an amino acid compound, a powerful chelating agent to remove inorganic debris (smear layer) and lubricate the canals in endodontics.

EDTA is used in 17% as neutralized solution (pH 8,5) to loosen up calcifications, remove smear layer, allow instrumentation (canals shaping) and debridement in a tight/calcified root canal towards the apex. For professional use only.

Instruction for use:
Irrigate with EDTA solution as needed for debridement during instrumentation.

- Fill syringe with EDTA solution (5 - 10 ml recommen-ded).
- Attach irrigation tip securely to syringe.
- Irrigate as desired.
- Remove and dispose of syringe and irrigating tip.

Precautions:
1. Read MSDS for material handling and safety information.
2. Irrigation should not be irritant for the apical area.
3. For professional use only.

Warnings:
1. Always replace cap after use.
2. Keep out of the reach of children.
3. Avoid contact with skin and eyes.

Storage:
Store in a room temperature, dry area, away from heat and direct sunlight.

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