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GB Description

Aspirating syringes are used in conjunction with a needle cartridge, Aspirating Syringes are available in Type CW and Type A, the syringes are reusable and are provided clean and supplied non-sterile.

Indication

Aspiarting syringes are indicated to be used in conjunction with anesthetic needles and anesthetic cartridges for injection of anesthetic soluutiions in the oral cavity.

Warnings

- To avoid needle stickes, re-cap needle after injectionhas been given.
- To avoid leakage during an injection, ensure that needle is straight as it penetrates the cartridge diaphragm.

Precaution

- Syringes must be sterilized prior to use.
- Hubs cannot be removed, do not attempt to remove hub from syringe for sterilization or for any reason as damage to the syringe will occur.

- Do not mix metals when sterilizing syringes, sterilizing syringes with non chrome-plated brass instruments may cause damage to syringe.
- Do not use cold sterilants, as they will react to the chrome plating of syringe.

Loading Instructions

- Retract piston rod fully.
- Insert 1.8 cc cartridge, plunger end first, into syringe barrel.
- Hold cartridge firmly in place, push piston or forward with a quick jab.
- Affix needle to aspirating syringe.
- Expel a few drops of solution to clear needle.
- Aspirate before injecting solution after inserting needle into tissue.

After positioning the needle, pull back on thumb ring to aspirate.

Unloading Instructions

- Remove needle from syringe.

- Withdraw piston rod to clear cartridge for removal
- Tilt syringe to expel to cartridge

Cleaning Instructions

- Remove all traces of blood, saliva and anesthesia solution.
- Remove paraffin that has accumulated on the harpoon.

Sterilization:

The instruments can be sterilized in designated container systems or general purpose sterilization containers individually packed in the standard polyethylene or Sterilization must be done by steam with fractionated pre-vacuum. Based on DIN EN 285 and DIN EN 554 is recommended at a temperature of 134 °C, a contact time of at least 5 minutes.

Effective steam sterilization can be achieved with the following cycle: Temperature cycle type minimum exposure time

Pre-vacuum to 134 °C - 137 °C for 5 minutes

To avoid damage the instrument, the temperature of 180 °C must not be exceeded.

Storage

Sterile, packaged instruments should be stored so that they are protected from dust, moisture, insects and vermin, and temperature extremes.

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MW Dental  Müller & Weygandt GmbH · Reichardsweide 40 · 63654 Büdingen/Germany · Tel.: +49 (0) 60 42 - 88 00 88