ULTRASUC SUPER

REF 000 787

**Inhalt/Content/Contenu/Contenuto/Zawartość/
Obsah/Tartalma: e 5 l**

- Ⓓ Absauganlagendesinfektion - Konzentrat
- ⒼⒷ Disinfection concentrate for suction systems
- Ⓕ Concentré de désinfection d'installations d'aspiration dentaires
- Ⓘ Disinfezione di impianti di aspirazione - Concentrato
- ⒫ Konzentrat do dezynfekcji ssaków
- ⒸⒶ Koncentrovaná dezinfekce do odsávacích zařízení
- Ⓗ Koncentrátum elszívórendszerek fertőtlenítésre

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☉ **Type of application:** For the cleaning and disinfection of dental suction systems (incl. saliva aspirators), amalgam separators and mouth rinsing basins.

Notes on application: The concentrate must be first diluted with water. Recommended application concentration: 2%. Add 20 ml concentrate with tap water to 1.000 ml and aspirate the solution through the suction hoses. Regularly, the suction system should be disinfected once or twice a day. The spittoon is disinfected by puring approx. 250 ml of the ready-to-use dilution. Do not mix with other disinfectants or cleansers! Always prepare ready-to-use solutions fresh and use immediately.

Exposure times: Bacteria and Candida albicans according to the VAH with high load as instrument disinfection:

2,0% 5 mins
1,0 % 60 mins.

Bacteria according to EN 13727 and EN 14561 (high load):
Candida albicans according to EN 13624 and EN 14562 (high load):
Enveloped viruses (i.e. HBV, HIV, HCV) acc. to RKI/DVV (with load):

3,0% 2 mins
3,0% 2 mins
2,0% 2 mins

100 g contain: 7.5 g Dimethyl dioctyl ammonium chloride, 0.6 g benzyl dimethyl dodecyl ammonium chloride

Spectrum of activity: Bactericidal (incl. MRSA) acc. to EN 13727, EN 14561, VAH, levurocidal (Candida albicans) acc. to EN 13624, EN 14562, VAH; limited virucidal (enveloped viruses such as, for example, HBV, HIV, HCV) acc. to DVV/RKI

Safety notes: H314 Causes severe skin burns and eye damage. H411 Toxic to aquatic life with long lasting effects. Avoid release to the environment. Wear protective gloves/eye protection. IF ON SKIN: Wash with plenty of water. IF IN EYES: Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention. Dispose of contents/container in accordance with all local, regional, national and international regulations. Contains N,N-Dimethyl-N-Dioctylammoniumchlorid, Kaliumhydroxid.

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