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AlproJet MixCup

For the preparation of AlproJet ready-to-use solutions for the maintenance and cleaning of dental aspiration systems.

1. User instructions

Please attend to the following instructions in order to guarantee a trouble-free, economic and safe work.

2. Symbols used on product information, labels and instructions for use

	Serial number XXXXXX (6 digits)
	Please follow the instruction for use
	CE-symbol for Medical Devices of class I
	Please note
	Order number
	Manufacturer, Company ALPRO MEDICAL GmbH, St. Georgen
	Date of production
	Machine readable code in HIBC format

3. Order data

- REF 3085 Multifunctional container with 2-fold adapter for th cleaning and maintenance of aspiration system
- REF 3088 Multifunctional container with 3-fold adapter for th cleaning and maintenance of aspiration system
- REF 3090 Connection adapter (2-fold) for AlproJet MixCup
- REF 3091 Connection adapter (3-fold) for AlproJet MixCup

4. Target group

This instruction for use is intended for all users (doctors, dentists and professionals in charge of the processing of medical devices as well as hygiene professionals) in medical and dental facilities.


5. Use the AlproJet MixCup only with the AlproJet series:

- REF 3104 **AlproJet-D**: Disinfecting liquid concentrate (5% application concentrations) for daily cleaning and care
- REF 3120 **AlproJet-DD**: Liquid concentrate (2% application concentrations) with extended disinfection effect for daily cleaning and care.
- REF 3106 **AlproJet-W**: Liquid concentrate (5% application concentrations) for weekly acidic contra-cleaning and care.

6. Application of AlproJet MixCup

2 L of a ready-to-use solution are required for one application. For this purpose dose 2 x **AlproJet** concentrate by using the dosage chamber of the dosage bottle and fill up in the **AlproJet MixCup** with water to 2 L of a ready-to-use solution. For this it is recommended to use 40 - 50 °C warm water. Afterwards aspirate 1 L of the ready-to-use solution through the aspiration tubes and pour the remaining 1 L into the spittoon bowl. Thereafter let take effect for at least 30 minutes and rinse the aspiration tubes and the spittoon bowl thoroughly with water just before starting work.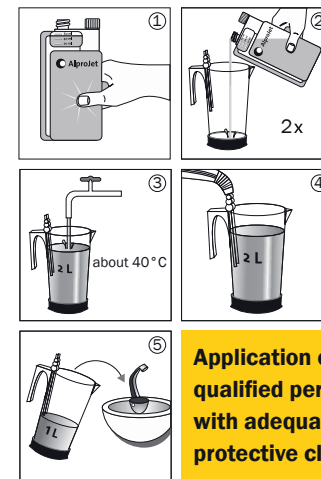

Application only by qualified personnel with adequate protective clothing.

⚠ When using the AlproJet MixCup with AlproJet-D, AlproJet-DD or AlproJet-W, please follow the instructions on the respective product label.

7. Maintenance

Visible soiling can be removed with AlproJet-D or AlproJet-DD and a plastic brush.

8. Report of incidents

Please report any incident that may occur in connection with the product to:

ALPRO MEDICAL GMBH

E-Mail: quality@alpro-medical.de

Tel.: +49 7725 9392-0

and the national authority

9. Disposal

The AlproJet MixCup must be disposed of with the residual waste, the packaging can be recycled.

10. Notes

Information on the preparation with regard to the work steps and all current work instructions, safety data sheets, instructions for use etc. can be found on our website:

www.alpro-medical.com

If you have any questions, please do not hesitate to contact us at www.alpro-medical.com or by phone +49 7725 9392-0. You are also welcome to contact our sales representatives by phone or e-mail.