3M™ Fittek™ Restoratives Dispenser

Restoratives Dispenser
Spenderpistole für Restaurativmaterialien
Pistolet distributeur de matériaux de restauration
Dispensador per composito
Dispensador de restaurador
Dispensador para material restaurador

Instructions for Use
 Gebrauchsinformation
 Mode d'emploi

Made in Germany

3M ESPE
 Dental Products
 2510 Conway Avenue
 St. Paul, MN 55144-100 USA
www.3M.com

3M, ESPE and Fittek are trademarks of 3M or
 3M Deutschland GmbH.

Used under license in Canada.

© 2020, 3M. All rights reserved.

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de 3M ou 3M Deutschland GmbH.

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3100017382/04

Examine the dispenser for visible contamination and repeat the pre-cleaning as necessary.
Automatic Cleaning and Thermal Disinfection
Automated washer-disinfectors that meet the following requirements are suitable for use:

• Sterilization cycle time is certified (e.g., DGHM or FDA approval) or CE marked.

• The restorative dispensers were wrapped once in disposable single sterilization pouches (e.g., Steriking, Wipak) and subjected to steam sterilization autoclave with the following parameters:

• Gravity displacement wrapped

• If available, a test cycle for thermal disinfection (AO value > 3000 or at least 5 min at 90 °C/194 °F, AO value > 5000 or at least 10 min at 90 °C/194 °F). The cycle that includes an adequate number of rinse cycles (take into account risk of disinfectant residues on the instruments when they are disinfected chemically).

• The air used for drying is filtered (cf. free, low microbiological and particle contamination).

The washer-disinfector is regularly serviced and tested.

Processing in an automated washer-disinfector

After the restorative dispensers have been pre-cleaned, place the restorative dispensers in the washer-disinfector.

Use an alkaline cleaning agent (e.g., Neodisher®/MediClean forte or Mediclean forte) or a equivalent agent.

Use an air-drying cycle (e.g., Neodisher®/MediClean forte or Mediclean forte) or a equivalent agent for automatic drying.

Deionized water (low germ, low endotoxin) is used for final rinsing.

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ENGLISH

Product Description

3M™ Fittek™ Restoratives Dispenser is a dispenser for the application of 3M and 3M ESPE dental materials in capsules.

Intended Use

Application of 3M dental materials from capsules.

Contraindications

None.

Intended Users

Educated dental professionals, i.e. general dentists, dental assistants/hygienists, who have theoretical and practical knowledge on usage of dental products.

Precautions

For Dental Personnel

• Insert the capsule firmly into the capsule holder when the cycle has completed.

• Examine the dispenser for visible impurities. If any impurities are visible, repeat the cycle in the washer-disinfector.

• Examine the dispenser again.

• Package the dispenser after removal from the device and dry in a sterilization pouch.

• After completing the thermal disinfection, carry out the sterilization process in accordance with the state Packaging and Sterilization (GDS) standard, and recognized test laboratory (Section 15 (5) MDPG (Medical Devices Act)), using the Miele G7830 CD washer-disinfector and the cleaning agent Neodisher®/Mediclean forte 0.2%.

• This product has been developed for the use of capsules from 3M or 3M ESPE.

• All the instructions for use must be followed. Any deviation from these instructions is the decision and the sole and exclusive responsibility of the dentist using the material.

Instructions for Use

• Turn the rotating capsule holder to the desired position for insertion of the capsule.

• Insert the capsule into the capsule holder until it audibly snaps into place.

• Apply the paste by slowly and steadily pressing the dispensing handle.

• Release the dispensing handle to withdraw the piston and to remove the capsule from the capsule holder.

Cleaning, Disinfection and Sterilization (Reprocessing)

Note: The Fittek Restoratives Dispenser cannot be taken apart.

Caution

Observe the following when selecting the cleaning and disinfectant agents that will be used:

• They should not contain the following substances: Oxidants (e.g., hydrogen peroxides), aldehydes, solvents, and oils.

• The pH value of the cleaning solution should be between pH 7.5 and 11.

• Do not expose the product to temperatures higher than 135 °C/275 °F.

• Follow manufacturer's instructions for concentrations, temperatures and times for cleaning and disinfection agents exactly. Use solvent solutions that have been freshly prepared with water of drinking water quality.

• Use only a soft, clean, and lint-free cloth for drying.

• Never use metal brushes or steel wool for cleaning. Always use a clean, soft cloth or cloth to remove all residues of the disinfectant.

• Use safe cleaning and disinfectant agents tested for efficacy and compatibility with the medical device (e.g., VAH approval, FDA clearance). Always observe all applicable legal and hygiene regulations for dental offices and/or hospitals.

Limitations on Reprocessing

The Fittek Restoratives Dispenser has been tested for 200 reprocessing cycles with no visible damage. This medical device is safe to use if it is reprocessed according to these instructions and no visible damage is seen. Inspect the device carefully before each use and discard if there is any damage.

General

The medical device is non-sterile when delivered and must be prepared and/or reprocessed according to instructions before every use.

Preparation for Decontamination

The medical device should be reprocessed as soon as possible after every use.

Reprocessing

Reprocessing of Fittek Restoratives Dispenser require the work steps pre-treatment, cleaning, disinfection and sterilization.

Cleaning and disinfecting the Fittek Restoratives Dispenser with an automated washer-disinfector (e.g., Neodisher®/Mediclean forte) is the most effective way to reprocess the device.

• The reprocessing procedure must be conducted before the initial use for every user. Cleaning and disinfecting should be done automatically.

Pre-Treatment

Pre-treatment must be done before every manual or automatic cleaning and disinfection. Gross impurities must be removed from the product immediately after use (within a maximum of 2 hours).

• Remove all visible impurities manually under running water (drinking water quality, temperature 22 °C–44 °C/71 °F–111 °F).

• Use a clean, soft, and dry disposal brush to rinse the depressions and recesses of the restorative dispensing device. The reprocessing procedure must be conducted before the initial use for every user. Cleaning and disinfecting should be done automatically.

Stabilization

Pre-treatment must be done before every manual or automatic cleaning and disinfection. Gross impurities must be removed from the product immediately after use (within a maximum of 2 hours).

• Remove all visible impurities manually under running water (drinking water quality, temperature 22 °C–44 °C/71 °F–111 °F).

• Use a clean, soft, and dry disposal brush to rinse the depressions and recesses of the restorative dispensing device. The reprocessing procedure must be conducted before the initial use for every user. Cleaning and disinfecting should be done automatically.

Decomposition

Pre-treatment must be done before every manual or automatic cleaning and disinfection. Gross impurities must be removed from the product immediately after use (within a maximum of 2 hours).

• Remove all visible impurities manually under running water (drinking water quality, temperature 22 °C–44 °C/71 °F–111 °F).

• Use a clean, soft, and dry disposal brush to rinse the depressions and recesses of the restorative dispensing device. The reprocessing procedure must be conducted before the initial use for every user. Cleaning and disinfecting should be done automatically.

Disinfection

Pre-treatment must be done before every manual or automatic cleaning and disinfection. Gross impurities must be removed from the product immediately after use (within a maximum of 2 hours).

• Remove all visible impurities manually under running water (drinking water quality, temperature 22 °C–44 °C/71 °F–111 °F).

• Use a clean, soft, and dry disposal brush to rinse the depressions and recesses of the restorative dispensing device. The reprocessing procedure must be conducted before the initial use for every user. Cleaning and disinfecting should be done automatically.

Sterilization

Pre-treatment must be done before every manual or automatic cleaning and disinfection. Gross impurities must be removed from the product immediately after use (within a maximum of 2 hours).

• Remove all visible impurities manually under running water (drinking water quality, temperature 22 °C–44 °C/71 °F–111 °F).

• Use a clean, soft, and dry disposal brush to rinse the depressions and recesses of the restorative dispensing device. The reprocessing procedure must be conducted before the initial use for every user. Cleaning and disinfecting should be done automatically.

Post-Treatment

Pre-treatment must be done before every manual or automatic cleaning and disinfection. Gross impurities must be removed from the product immediately after use (within a maximum of 2 hours).

• Remove all visible impurities manually under running water (drinking water quality, temperature 22 °C–44 °C/71 °F–111 °F).

• Use a clean, soft, and dry disposal brush to rinse the depressions and recesses of the restorative dispensing device. The reprocessing procedure must be conducted before the initial use for every user. Cleaning and disinfecting should be done automatically.

Reprocessing

Reprocessing of Fittek Restoratives Dispenser require the work steps pre-treatment, cleaning, disinfection and sterilization.

Cleaning and disinfecting the Fittek Restoratives Dispenser with an automated washer-disinfector (e.g., Neodisher®/Mediclean forte) is the most effective way to reprocess the device.

• The reprocessing procedure must be conducted before the initial use for every user. Cleaning and disinfecting should be done automatically.

Stabilization

Pre-treatment must be done before every manual or automatic cleaning and disinfection. Gross impurities must be removed from the product immediately after use (within a maximum of 2 hours).

• Remove all visible impurities manually under running water (drinking water quality, temperature 22 °C–44 °C/71 °F–111 °F).

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3M, ESPE and Filtek sont des marques de commerce de 3M ou 3M Deutschland GmbH. Used under license in Canada. © 2020, 3M. All rights reserved.
3M, ESPE and Filtek are trademarks of 3M or 3M Deutschland GmbH. Utilisées sous licence au Canada. © 2020, 3M. Tous droits réservés.
3M, ESPE and Filtek are trademarks of 3M or