SILANISIERUNGS- LIQUID

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REF 000 910

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GB Description:

The Silanization Liquid is a treatment agent for surfaces and ensures a durable chemical bond between metal composites, ceramic composites, and composite composites after prior silicatisation.

The contained silane enables a permanent and gap-free connection between the above-mentioned silicatised surfaces and acrylate-based UV-curable composites or two-component composites. For professional users only.

GB Instructions for use:

For intraoral use, place rubber dam. The surface to be treated is first silicatised using the Rocatec System or the CoJet System. Always work with clean, grease- and dust-free working materials. Do not blow off the wetted surface with compressed air or by mouth.

In case of contact with the fingers, the surface must be silicatised again. Observe the respective instructions for use of the silicatisation process as well as the opaque, composite and bonder. Do not use as a bonding agent for fibre-reinforced root canal posts or for veneer fixation in the anterior region!

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Gefahr

Procedure:

- Place the Silanization Liquid in a fat-free bowl. The surface to be veneered is completely wetted once with a clean brush that is to be used for this purpose only. Repeated application of the liquid reduces the adhesive effect.
- The liquid evaporates quickly. To complete the chemical reaction on the treated surface and thus achieve an optimal adhesive bond between the metal surface and the opaque, it must be left to dry for at least 5 minutes.
- Leave to dry for 30 seconds for interoral application. The surfaces must be completely dry before opaque or composite is applied.
- After the drying time, continue according to the instructions for use of the opaque, composite or bonder used.

Intolerances:

If allergic reactions occur, discontinue using the Silanization Liquid and remove the silane completely.

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