# Bluephase® Style


## Start-up

- 1 ✓ Unpack handpiece  
✓ Detach the light guide by pulling it out  
✓ Clean them (see Instructions for Use)
- 2 ✓ Unpack charging base  
✓ Slide connector plug into its socket
- 3 ✓ Connect the mains cable to the power supply unit and supply mains

- 4 ✓ Slide battery straight into the handpiece until you hear and feel it click into place.
- 5 ✓ Gently place the handpiece in the corresponding rest in the charging base without using any force.


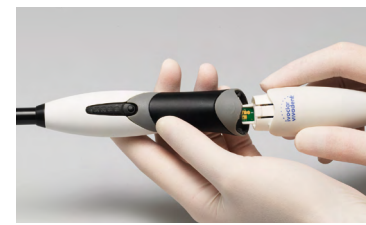
**i** Mount the anti-glare cone on the light probe.

**i** If the battery is fully discharged, the charging time is approx. 2 hours

## 6 Operation

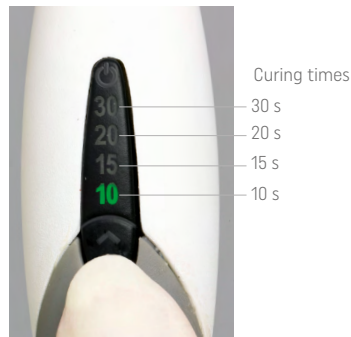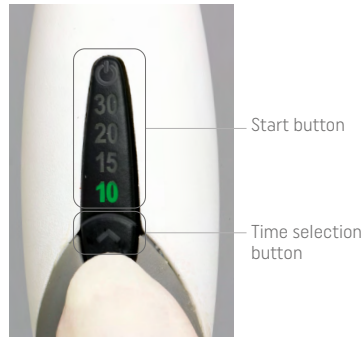Curing times		HIGH POWER 1,200 mW/cm <sup>2</sup> ± 10%
Restorative materials	Composites • 2 mm <sup>1)</sup> IPS Empress Direct / IPS Empress Direct Effect Tetric EvoCeram / Tetric EvoFlow Tetric / Tetric Basic White	10 seconds
	• 4 mm <sup>2)</sup> Tetric EvoCeram Bulk Fill / Tetric EvoFlow Bulk Fill Tetric Basic White	10 seconds
	Heliomolar / Heliomolar HB / Heliomolar Flow All conventional composites <sup>3)</sup>	15 seconds
	Compomers <sup>3)</sup> Compoglass F / Compoglass Flow	20 seconds
Indirect restorations / cementation materials	Variolink Esthetic LC <sup>4)</sup> / Variolink Esthetic DC <sup>4)</sup>	per mm ceramic: 10 seconds per surface
	Multilink Automix <sup>5)</sup> / SpeedCEM Plus <sup>5)</sup>	per mm ceramic: 20 seconds per surface
Adhesives	AdheSE / Adhese Universal / Excite F / Excite F DSC Heliobond / Syntac	10 seconds
Temporary materials	Telio CS Link	10 seconds per surface
	Telio CS Inlay/Onlay	10 seconds
	Telio Stains	10 seconds
	Telio Add-On Flow	15 seconds
Miscellaneous	Helioseal / Helioseal F / Helioseal Clear Heliosit Orthodontic	20 seconds 10 seconds
	IPS Empress Direct Color IPS Empress Direct Opaque	20 seconds 20 seconds
	MultiCore Flow / Multicore HB	20 seconds

1) Applies to a maximum layer thickness of 2 mm and provided that the Instructions for Use of the respective material do not state any other recommendation (might be the case e.g. with dentin shades)  
2) Applies to a maximum layer thickness of 4 mm and provided that the Instructions for Use of the respective material do not state any other recommendation (might be the case e.g. with dentin shades)

3) Applies to a maximum layer thickness of 3 mm  
4) Applies to light-curing  
5) Applies to dual-curing

## What if ...?

If any of these errors persist after trying their corresponding rectification, please contact your dealer or your local Service Centre.

Indicator	Causes	Rectification
PRESSING BATTERY BUTTON DOES NOT WORK	 Snapper worn out (too many removals of the battery)	Replace battery
	Snapper broken	<b>i</b> Note: Battery can be used as long as it lasts in the handpiece
BATTERY LOOSE	 Snapper worn out (too many removals of the battery)	Replace battery
	Snapper broken	<b>i</b> Note: Battery can be used as long as it lasts in the handpiece
BATTERY IS NOT LOADING	Battery deeply discharged	Charge battery without handpiece in charging base (approx. 1 h)
	Charging base or power supply unit defective	Replace charging base or power supply unit
	Battery defective	Replace battery

- Materials, the polymerization of which is activated outside the wavelength range of 385–515 nm (no materials known to date). If you are not sure about certain products, please ask the manufacturer of the corresponding material.
- Portable and mobile high-frequency communication devices may interfere with medical equipment. The use of mobile phones during operation is not allowed.

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