PERMAPLAST LH CERAMIC

Stand: 27/01/20

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Suitable for all cavities.

Side Effects

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

Preparation of Tooth and Colour Matching

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation and colour matching.

Ascertain the tooth shade while teeth are still moist and select the appropriate M+W Permaplast LH Ceramic shade.

M+W Permaplast LH Ceramic is shaded according to VITA®-Shades.

Cavity Preparation

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction.

For pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

Condition and apply a bonding agent according to the manufacturer instructions. The total etch technique with ETCHING GEL and subsequent application of a light cure bonding agent is recommended. Otherwise use a

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self etching bonding agent before application of the M+W Permaplast LH Ceramic.

Application and Curing of M+W Permaplast LH Ceramic

Apply M+W Permaplast LH Ceramic in the selected shade. M+W Permaplast LH Ceramic is easy to model. Place it by using a suitable instrument.

Transparent matrix strips may be used.

In case of deep and extended cavities best results are obtained with a build-up of a base liner using a light cure resin modified cement or a light cure flowable composite, followed by the application of M+W Permaplast LH Ceramic.

In case of small cavities M+W Permaplast LH Ceramic can be applied directly. For an optimum result apply M+W Permaplast LH Ceramic in **layers of max. 2 mm**. Light cure each increment separately.

By using a polymerization unit with a light intensity of at least 1000mW/cm² cure each increment as follows:

Lighter shades (e.g. A1, A2, C2) **20 seconds**

Darker shades (e.g. A3.5, B3) **30 seconds**

Finishing

Contour with fine diamonds, stones or burs. Polish to high gloss with discs or rubber points. Interproximal finishing is accomplished by fine grit finishing strips.

Storage

Do not store above 25°C (77°F). Protect from direct sunlight. Do not use after expiry date.

Close syringe immediately after use to avoid exposure to light.

Cavifils are for single use only.

Additional Notes

- Do not use any resin to adjust viscosity of composite restorative material.
- Contact of resin pastes with skin should be avoided, especially by anyone having known resin allergies.
- VITA® is a registered trade mark of the VITA-Zahnfabrik, Bad Säckingen, Germany.

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Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior Dental Materials GmbH product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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Manufacturer: P.L. Superior Dental Materials GmbH · 22607 Hamburg · Germany