- D** FARBWALZENTAUSCH
GB CHANGING THE INKING ROLLERS
F CHANGEMENT DU ROULEAU ENCREUR

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GB Please press the keys marked with 1 together and pull the roller holder forwards (Fig. E)

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GB Now press the arms on the inside of the inking roller together, pull the arms upwards and remove the inking roller together with the arms. Change the inking roller. Insert the roller again, allow the arms to engage and reclose the roller holder.

If the contact pressure of the roller is not sufficient, you can shift the printing unit (see figure H) forward using a Philips screwdriver.

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GB Instructions for use M+W labeller

A system for labelling sterile goods packaging, equipped with double self-adhesive labels.

Data settings:

You can set the following data:

- Name of employee up to 5 digits
- Lot up to 7 digits
- See illustration
- Now insert the label strip behind the aluminium transport roller.

To set the data, please proceed as follows: Setting text characters

- move the dial to the right into the desired position
- now rotate the dial to enter the desired letter

Please note that the following settings are to be taken for entering letters:

If you would like to use the letter T, please rotate the dial to letter G (see example below). You can find the list of letters at the head of the labeller (silver label)

A	B	C	D	E	F	G	H	I	J	K	L	M
N	O	P	Q	R	S	T	U	V	W	X	Y	Z

Example:
Your name: T I N A
Select letters: G V A N

Labelling:
Press lever together and release without putting the device down. This transfers pressure to the label, and the label is generated.

Place the device on the desired position of the packaging/container and drag slightly without pressing the lever again.

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GB Please pull the levers marked with a 2 (left and right Fig. A) backwards. This will open the base assembly 1 (Fig. A)

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GB Insert a new roll of labels (Fig. B). Please make sure that the back of the labels faces upwards. Then pull the label strip out from the labeller (15cm) and close the base assembly 1 until it engages. Remove the labels on the 15cm label strip.

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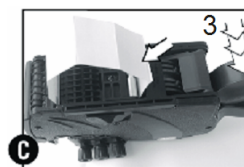
GB Now insert the label strip (Fig. C). Activate lever 3 repeatedly until the label strip feeds in automatically. After repeated activation, you will produce the first labels which will align automatically

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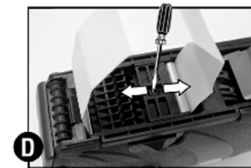
GB Should automatic alignment fail, please use a Philips screwdriver to loosen screw slightly. You can now adjust the sliding plate. After adjustment, you can retighten the cross-headed screw

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