3M ESPE**KetacTM Molar AplicapTM****Glass Ionomer Restorative Material – Radiopaque****Glasionomer Füllungsmaterial – Röntgensichtbar****Matiere d'obturation en verre ionomère – Radiopaque****Materiale per otturazione vetro-ionomero – Radiopaco****Material de obturación de vidrio ionómero – Radiopaco****Material de obturación de vidro ionómero – Radiopaco****Glasionomer vulmateriaal – Radiopaak****Υαλοτονομερικός κονίος για εμφρόξεις – Ακτινούχο****Glasionomer fyllningsmaterial – Röntgenopak****Lasi-ionomeerityeaine – Röntgenpaakki****Glasionomer-fylndingsmateriale – Radiopaque****Glassionomer fyllingsmateriale – Med röntgenkontrast**

Instructions for Use
Gebrauchsinformation
Mode d'emploi
Informazioni per l'uso
Información de uso
Instruções de uso

Gebruiksinfo
Οδηγίες χρήσης
Bruksanvisning
Käytöinno
Brugsanvisning
Bruksinformasjons

301000934601

3M Deutschland GmbH
Dental Products
Carl-Schurz-Str. 1
41453 Neuss – Germany

3M ESPE Customer Care/MSDS Information:
U.S.A. 1-800-634-2249 and Canada 1-888-363-3685.

3M, ESPE, Aplicap, CapMix, Ketac, RotoMix and ScotchbondTM are trademarks of 3M or 3M Deutschland GmbH.
Used under license in Canada. © 2020, 3M. All rights reserved.

ENGLISH**Product Description**

KetacTM Molar AplicapTM is a radiopaque bulk fill glass ionomer cement in capsules. On account of its chemical adhesion to the dental enamel and dentin, a gentle preparation and an especially tight seal at the margins is possible. The product is available in various shades; it can be applied without lining and releases fluoride ions. The dispensable quantity of a capsule is at least 0.14 ml.

These Instructions for Use should be kept for the duration of product use.

For details on all mentioned products, please refer to the respective Instructions for Use.

Indications

- Linings for single-surface and multiple-surface composite fillings
- Cone build-up prior to crown placement
- Primary tooth fillings
- Single-surface fillings in non-oclusal load-bearing areas
- Stress bearing Class II restorations when the isthmus is less than half of the intercuspal distance and with at least one occlusal contact on enamel
- Cervical fillings, if aesthetics is not the prime consideration
- Zahnfahlungen bei nachrangiger Ästhetik
- Ein- und mehrflächige temporäre Füllungen

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► Mix the capsule at approx. 4,300 rpm in a high-frequency mixing device such as CapMix or the RotoMix rotating mixer. In doing so, keep the application nozzle shut.

► Apply the capsule into the AplicapTM Applier and open the application nozzle.

► Apply the glass ionomer cement in the cavity and shape the filling.

– Ensure that there is no contamination from water, saliva, etc., for the entire duration of application and shaping; the working environment must be kept dry.

Applikation

► Place the AplicapTM Activator on a sturdy surface and insert the Aplicap capsule into the activator. In doing so, keep the application nozzle shut.

– Both the Aplicap Activator and the Applier are color-coded in orange.

► Using the ball of your hands, depress the activator lever firmly and completely to the stop and hold it down for 2 to 4 seconds.

– Firmly and completely depressing the lever to the stop and holding it in this position is the only way to ensure that all of the liquid is pressed into the powder. If inadequate pressure is applied or if the lever is not depressed completely, it is possible that too little liquid will be mixed with the powder, increasing the viscosity and changing the product characteristics.

► Mix the capsule at approx. 4,300 rpm in a high-frequency mixing device such as CapMix or the RotoMix rotating mixer. In doing so, keep the application nozzle shut.

► Apply the capsule into the AplicapTM Applier and open the application nozzle.

– Ensure that there is no contamination from water, saliva, etc., for the entire duration of application and shaping; the working environment must be kept dry.

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► Mix the capsule at approx. 4,300 rpm in a high-frequency mixing device such as CapMix or the RotoMix rotating mixer. In doing so, keep the application nozzle shut.

► Apply the capsule into the AplicapTM Applier and open the application nozzle.

– Ensure that there is no contamination from water, saliva, etc., for the entire duration of application and shaping; the working environment must be washed off with cold water before setting.

Füllung Protection

Applying a protective coat to the filling is not mandatory. If a protective coat is desired, vaseline and/or cotton rolls can be used as follows:

– Immediately after shaping, apply vaseline to all free surfaces of the glass ionomer cement.

► Further finishing with Arkansas stones, fine-grain diamonds, abrasive discs with successively decreasing grain size, or silicone polishers should be started 4:30 minutes after start of mixing at the earliest.

► Optionally apply vaseline again for surface sealing.

Stand der Information November 2019

Storage and Shelflife

Store the capsules in the blister packs.
Use within 1 month maximum once the blister pack has been opened!
Store the product at 15–25 °C/59–77 °F.

Do not use after the expiry date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

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